Art. 420600 = FFP2 NR-D #22020
Art. 420700 = FFP2-V NR D #22021
Art. 420900 = FFP3-V NR D #22231
EN 149:2001+A1:2009

DB: 1796648
DB: 1824063
DB: 1952738

DK - BRUGSANVISNING

Filtrerende engangsmaske

VIGTIGT:

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GB - INSTRUCTIONS FOR USE

Single-use filtering face-piece respirators

IMPORTANT:

Single-use respirators are certified in accordance with the European PPE Regulations 2016/425 on safety requirements for personal protective equipment and classified to one of three categories: FFP1 FFP2 or FFP3. Before reading the following information, check the category to which the respirator belongs-this is indicated on the packaging and on the respirator.

If the mask is marked D, it has passed the dolomite clogging test and suitable for use in very dusty environment.

It is the user's responsibility to ensure that the respirator provides the necessary level of protection for the type and concentration of the contaminant(s) in the area where the respirator is intended for use.

DIRECTIONS FOR USE:

- Fit the respirator and check leak-tightness BEFORE entering the contaminated area.
- Wear the respirator for the whole duration of exposure to contaminants.
- Use the respirator in accordance with applicable health and safety regulations.

Discard the respirator and replace with a new one if:

- the respirator is removed while in a contaminated area
- excessive clogging of the respirator causes breathing difficulty or discomfort
- the respirator becomes damaged
- (for respirators protecting against vapours)-the smell of vapours present becomes detectable.
- Leave the contaminated area if dizziness, irritation or other distress occurs.
- Only for single use, No maintenance necessary. Do not store and/or re-use after single use.
- Discard the respirator after single use (one shift, maximum).
- Keep un-used respirators in their closed box and store in a dry non-contaminated area between -20°C and +40°C at relative humidity under 75%.

Notified Body: BSI Product Services (0086), Kitemark Court, Davy Ave, Knowlhill, Milton Keynes, MK5 8PP, UK.

FITTING INSTRUCTIONS:

1. Cup the respirator in the hand, allowing the head-bands to hang freely.
2. Hold the respirator under chin with the nose-piece facing outwards.
3. Place the lower head-band around the neck below the ears. Holding the respirator against the face with one hand, place the top head-band above the ears, around the crown of the head.
4. Respirators with adjustable head-bands only: while maintaining the position of the respirator, adjust tension by pulling tab of each head-band, (Tension may be decreased by pushing out on the back of the buckle).
5. Mould the nose area to the shape of the face, running the fingertips of both hands from the top of the nose-piece down both sides while pressing inward.

CHECK FOR LEAK-TIGHTNESS (FACIAL FIT) AS FOLLOWS:

- place both hands over the respirator and exhale sharply
- positive pressure should be inside the respirator
- if you feel air escaping around the edges, re-adjust the respirator by tightening the nose-piece and/or increasing tension of the head-bands
- repeat the procedure until satisfactory face-fit has been achieved.

IMPORTANT: In the case of respirators with non-adjustable head-bands-if you feel that the head-bands are too loose, you can increase tension by tying a small knot in the head-band.

LIMITATIONS TO USE:

Do not use these respirators or enter or stay in an area where:

- Oxygen concentration is less than 17% (UK-19%)
- Contaminants or their concentrations are unknown or immediately dangerous to life or health
- Particulate concentrations exceed levels fixed by applicable health and safety regulations or MAC x NPF-whichever is lower, [Nominal protection factors: 4.5 for FFP1, 12.5 for FFP2 and, 50 for FFP3]
NPF: nominal protection factor
MAC: maximum admissible concentration
- Gases and/or vapours are present-except for respirators designated as protecting against vapours, in which case vapour concentrations should not exceed NPF. Respirators protecting against organic vapours should not be used as protection against acid vapours and vice-versa.

WARNING:

Do not use for fire-fighting. These respirators do not supply oxygen. Do not use in oxygen-deficient atmospheres-e.g. tanks or other poorly-ventilated areas (see 'Limitations to use'). Do not use in explosive atmospheres. For use only by trained and qualified personnel. Requirements for leak-tightness are unlikely to be achieved if the respirator is worn against a beard or facial stubble. When transporting this product, retain in the original packaging and keep away from mechanical and chemical hazards.

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Testinstitutt: BSI Product Services (0086), Kitemark Court, Davy Ave, Knowlhill, Milton Keynes, MK5 8PP, UK.

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